



AR Configuration Utility (ARCU) User's Guide

Note the following details of the code protection feature on Microchip devices:

- Microchip products meet the specification contained in their particular Microchip Data Sheet.
- Microchip believes that its family of products is one of the most secure families of its kind on the market today, when used in the intended manner and under normal conditions.
- There are dishonest and possibly illegal methods used to breach the code protection feature. All of these methods, to our knowledge, require using the Microchip products in a manner outside the operating specifications contained in Microchip's Data Sheets. Most likely, the person doing so is engaged in theft of intellectual property.
- Microchip is willing to work with the customer who is concerned about the integrity of their code.
- Neither Microchip nor any other semiconductor manufacturer can guarantee the security of their code. Code protection does not mean that we are guaranteeing the product as "unbreakable."

Code protection is constantly evolving. We at Microchip are committed to continuously improving the code protection features of our products. Attempts to break Microchip's code protection feature may be a violation of the Digital Millennium Copyright Act. If such acts allow unauthorized access to your software or other copyrighted work, you may have a right to sue for relief under that Act.

Information contained in this publication regarding device applications and the like is provided only for your convenience and may be superseded by updates. It is your responsibility to ensure that your application meets with your specifications. MICROCHIP MAKES NO REPRESENTATIONS OR WARRANTIES OF ANY KIND WHETHER EXPRESS OR IMPLIED, WRITTEN OR ORAL, STATUTORY OR OTHERWISE, RELATED TO THE INFORMATION, INCLUDING BUT NOT LIMITED TO ITS CONDITION, QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR PURPOSE. Microchip disclaims all liability arising from this information and its use. Use of Microchip devices in life support and/or safety applications is entirely at the buyer's risk, and the buyer agrees to defend, indemnify and hold harmless Microchip from any and all damages, claims, suits, or expenses resulting from such use. No licenses are conveyed, implicitly or otherwise, under any Microchip intellectual property rights.

Trademarks

The Microchip name and logo, the Microchip logo, dsPIC, KEELOQ, KEELOQ logo, MPLAB, PIC, PICmicro, PICSTART, PIC³² logo, rPIC and UNI/O are registered trademarks of Microchip Technology Incorporated in the U.S.A. and other countries.

FilterLab, Hampshire, HI-TECH C, Linear Active Thermistor, MXDEV, MXLAB, SEEVAL and The Embedded Control Solutions Company are registered trademarks of Microchip Technology Incorporated in the U.S.A.

Analog-for-the-Digital Age, Application Maestro, chipKIT, chipKIT logo, CodeGuard, dsPICDEM, dsPICDEM.net, dsPICworks, dsSPEAK, ECAN, ECONOMONITOR, FanSense, HI-TIDE, In-Circuit Serial Programming, ICSP, Mindi, MiWi, MPASM, MPLAB Certified logo, MPLIB, MPLINK, mTouch, Omniscent Code Generation, PICC, PICC-18, PICDEM, PICDEM.net, PICKit, PICtail, REAL ICE, rLAB, Select Mode, Total Endurance, TSHARC, UniWinDriver, WiperLock and ZENA are trademarks of Microchip Technology Incorporated in the U.S.A. and other countries.

SQTP is a service mark of Microchip Technology Incorporated in the U.S.A.

All other trademarks mentioned herein are property of their respective companies.

© 2011-2012, Microchip Technology Incorporated, Printed in the U.S.A., All Rights Reserved.

 Printed on recycled paper.

ISBN:

Microchip received ISO/TS-16949:2009 certification for its worldwide headquarters, design and wafer fabrication facilities in Chandler and Tempe, Arizona; Gresham, Oregon and design centers in California and India. The Company's quality system processes and procedures are for its PIC® MCUs and dsPIC® DSCs, KEELOQ® code hopping devices, Serial EEPROMs, microperipherals, nonvolatile memory and analog products. In addition, Microchip's quality system for the design and manufacture of development systems is ISO 9001:2000 certified.

**QUALITY MANAGEMENT SYSTEM
CERTIFIED BY DNV
= ISO/TS 16949 =**

AR Configuration Utility (ARCU) User's Guide

**Object of Declaration: AR1010, AR1011, AR1020, AR1021, DV102011, DV102012,
AR1100, AR1100BRD**

EU Declaration of Conformity

Manufacturer: Microchip Technology Inc.
2355 W. Chandler Blvd.
Chandler, Arizona, 85224-6199
USA

This declaration of conformity is issued by the manufacturer.

The development/evaluation tool is designed to be used for research and development in a laboratory environment. This development/evaluation tool is not intended to be a finished appliance, nor is it intended for incorporation into finished appliances that are made commercially available as single functional units to end users. This development/evaluation tool complies with EU EMC Directive 2004/108/EC and as supported by the European Commission's Guide for the EMC Directive 2004/108/EC (8th February 2010).


This development/evaluation tool complies with EU RoHS2 Directive 2011/65/EU.

For information regarding the exclusive, limited warranties applicable to Microchip products, please see Microchip's standard terms and conditions of sale, which are printed on our sales documentation and available at www.microchip.com.

Signed for and on behalf of Microchip Technology Inc. at Chandler, Arizona, USA



Derek Carlson
VP Development Tools

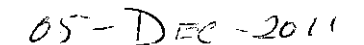

Date

AR Configuration Utility (ARCU) User's Guide

NOTES:



AR CONFIGURATION UTILITY (ARCU) USER'S GUIDE

Table of Contents

Declaration of Conformity	3
Preface	7
Introduction	7
Document Layout.....	7
Conventions Used in this Guide	8
Recommended Reading	9
The Microchip Web Site.....	9
Customer Support.....	9
Document Revision History	9
Chapter 1: Installation.....	11
Chapter 2: Getting Started.....	13
Chapter 3: AR Configuration Utility Layout	15

AR Configuration Utility (ARCU) User's Guide

NOTES:

Preface

NOTICE TO CUSTOMERS

All documentation becomes dated, and this manual is no exception. Microchip tools and documentation are constantly evolving to meet customer needs, so some actual dialogs and/or tool descriptions may differ from those in this document. Please refer to our web site (www.microchip.com) to obtain the latest documentation available.

Documents are identified with a “DS” number. This number is located on the bottom of each page, in front of the page number. The numbering convention for the DS number is “DSXXXXA”, where “XXXX” is the document number and “A” is the revision level of the document.

For the most up-to-date information on development tools, see the MPLAB® IDE on-line help. Select the Help menu, and then Topics to open a list of available online help files.

INTRODUCTION

This chapter contains general information that will be useful to know before using the AR Configuration Utility (ARCU). Items discussed in this chapter include:

- Document Layout
- Conventions Used in this Guide
- Recommended Reading
- The Microchip Web Site
- Customer Support
- Document Revision History

DOCUMENT LAYOUT

This document describes the installation and use of the AR Configuration Utility (ARCU). The Microchip ARCU is a Windows®-based graphical user interface that can be used to configure, calibrate, test and demonstrate both the AR1000 and AR1100 touch screen controllers. Additionally, the AR1100 firmware can be updated (re-flashed) via the ARCU. The manual layout is as follows:

- **Chapter 1. Installation**
- **Chapter 2. Getting Started**
- **Chapter 3. AR Configuration Utility Layout**

AR Configuration Utility (ARCU) User's Guide

CONVENTIONS USED IN THIS GUIDE

This manual uses the following documentation conventions:

DOCUMENTATION CONVENTIONS

Description	Represents	Examples
Arial font:		
Italic characters	Referenced books	<i>MPLAB® IDE User's Guide</i>
	Emphasized text	...is the <i>only</i> compiler...
Initial caps	A window	the Output window
	A dialog	the Settings dialog
	A menu selection	select Enable Programmer
Quotes	A field name in a window or dialog	"Save project before build"
Underlined, italic text with right angle bracket	A menu path	<u><i>File>Save</i></u>
Bold characters	A dialog button	Click OK
	A tab	Click the Power tab
N'Rnnnn	A number in verilog format, where N is the total number of digits, R is the radix and n is a digit.	4'b0010, 2'hF1
Text in angle brackets < >	A key on the keyboard	Press <Enter>, <F1>
Courier New font:		
Plain Courier New	Sample source code	#define START
	Filenames	autoexec.bat
	File paths	c:\mcc18\h
	Keywords	_asm, _endasm, static
	Command-line options	-Opa+, -Opa-
	Bit values	0, 1
	Constants	0xFF, 'A'
Italic Courier New	A variable argument	<i>file.o</i> , where <i>file</i> can be any valid filename
Square brackets []	Optional arguments	mcc18 [options] <i>file</i> [options]
Curly brackets and pipe character: { }	Choice of mutually exclusive arguments; an OR selection	errorlevel {0 1}
Ellipses...	Replaces repeated text	var_name [, var_name...]
	Represents code supplied by user	void main (void) { ... }

RECOMMENDED READING

This user's guide describes how to use the AR Configuration Utility (ARCU). Other useful documents are listed below. The following Microchip documents are available and recommended as supplemental reference resources.

Readme for AR Configuration Utility (ARCU)

For the latest information on using the AR Configuration Utility (ARCU), read the "Readme.txt" file (an ASCII text file) in the main installation directory. The Readme file contains updated information and known issues that may not be included in this user's guide.

Readme Files

For the latest information on using other tools, read the tool-specific Readme files in the Readmes subdirectory of the MPLAB® IDE installation directory. The Readme files contain updated information and known issues that may not be included in this user's guide.

THE MICROCHIP WEB SITE

Microchip provides online support via our web site at www.microchip.com. This web site is used as a means to make files and information easily available to customers. Accessible by using your favorite Internet browser, the web site contains the following information:

- **Product Support** – Data sheets and errata, application notes and sample programs, design resources, user's guides and hardware support documents, latest software releases and archived software
- **General Technical Support** – Frequently Asked Questions (FAQs), technical support requests, online discussion groups, Microchip consultant program member listing
- **Business of Microchip** – Product selector and ordering guides, latest Microchip press releases, listing of seminars and events, listings of Microchip sales offices, distributors and factory representatives

CUSTOMER SUPPORT

Users of Microchip products can receive assistance through several channels:

- Distributor or Representative
- Local Sales Office
- Field Application Engineer (FAE)
- Technical Support

Customers should contact their distributor, representative or field application engineer (FAE) for support. Local sales offices are also available to help customers. A listing of sales offices and locations is included in the back of this document.

Technical support is available through the web site at: <http://support.microchip.com>

AR Configuration Utility (ARCU) User's Guide

DOCUMENT REVISION HISTORY

Revision A (August 2011)

- Initial Release of this Document.

Revision B (July 2012)

- Added Declaration of Conformity; Added functional support for the AR1011 and AR1021 controllers; Changed the title from “AR1100 Configuration Utility” into “AR Configuration Utility”; Updated Figures 2-1, 3-1, 3-2, 3-4, 3-6, 3-7, 3-9, 3-13, 3-14 and 3-15; Updated section 3.1; Other minor corrections.

Chapter 1. Installation

The Microchip AR Configuration Utility (ARCU) is a Windows-based graphical user interface that can be used to configure, calibrate, test and demonstrate both the AR1000 and AR1100 series touch screen controllers. Additionally, the AR1100 firmware can be updated (re-flashed) via ARCU.

Note: This version of the utility (revision 2.20) is the correct one for the AR1011 and AR1021 part numbers. Previous revisions of the utility will not work with these two new devices.

The AR1000 can use either the I²C™, SPI or USART based communication protocols, while the AR1100 can use USB or RS-232 (serial). To facilitate AR1000 communications, Microchip's USB-based PICKIT™ Serial Analyzer is employed to translate communications from the host PC to the desired embedded protocol, and visa versa. Provided the correct hardware is used, changing protocols in the ARCU is achieved via a wizard or a simple options selection.

1.1 INSTALLATION

If installing from a CD, insert the CD into your PC and follow the installation instructions. If installing from a zip file, double click on the file, unzip it and follow the installation instructions.

Note: Recommended operating systems include Windows® XP 32- and 64-bit, Windows 7 32- and 64-bit. It will require .NET 2.0 to run this application. Some installations of .NET 4.0 do not include the .NET 2.0 framework. In that case, you will need to download either the 2.0 or 3.5 (Service Pack 1) version of .NET.

AR Configuration Utility (ARCU) User's Guide

NOTES:

Chapter 2. Getting Started

2.1 GETTING STARTED

- If using either the AR1000 or AR1100 Development kits, follow the Connections step in the corresponding Quick Start Guide.
- If using the AR1000, connect the PICKit Serial Analyzer to the PC, the AR1000 board to the PICKit Serial Analyzer, and a sensor to the AR1000 board.
Note: Do not use a USB hub.
- If using the AR1100, connect either the USB or serial communications cable to the AR1100 board and to the PC. Attach a sensor to the AR1100 board. If using a serial connection, apply a 5V power supply to the serial leads (see the AR1100 PCB User's Guide for details).

Start the ARCU by double clicking the desktop icon. After the splash screen disappears, a welcome screen will appear:

FIGURE 2-1: WELCOME SCREEN



If this is the first time the ARCU has been used, select the Start Wizard icon to step you through the configuration process. Follow the steps outlined in the Wizard. If using the AR1020 (I²C, SPI) Development kit or the AR1100 Development kit (USB, Serial), have the calibration template printed and ready to use.

AR Configuration Utility (ARCU) User's Guide

Note: If you are using the AR1000 or the AR1020 Development kit, the Configuration Wizard may initially ask you to update the PICKit Serial Analyzer firmware. If it does, cancel out of the Wizard and press the **Update PKSA Firmware** button on the Commands panel. Allow the firmware to update, and then restart the Wizard.

After the Wizard is complete, you should have established communications with the AR1000/AR1100 and you will be taken to the first tab of the ARCU – the **Configuration** tab. The Transactions window (automatically displayed) will show recent communications between the ARCU and the controller.

If there were errors, they will appear in red text in the Transaction window. Communication errors are generally the result of physical connection or jumper issues. If you have errors, check your connections and restart the wizard by pressing the **Wizard** button on the Commands panel (Basic view) or in the Ribbon Tools Group (Diagnostic view).

After you have successfully completed the Wizard, touch the sensor and observe the touch reports scrolling in the Transactions window.


Chapter 3. AR Configuration Utility Layout

AR CONFIGURATION UTILITY LAYOUT

The ARCU has four tabs, or pages:

- The Configuration Tab
- The Scribble Demo
- The Equalizer Demo
- The P.O.S. Demo

FIGURE 3-1: TABS

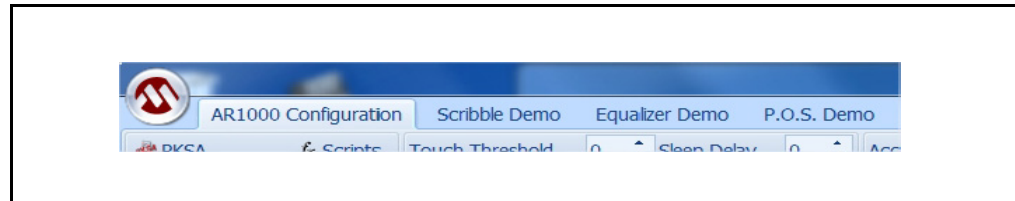


Note: The text of the **Configuration** tab and the title bar will vary between AR1000 and AR1100, depending on how the ARCU is configured.

The **Configuration** tab is used to test, calibrate and configure the AR1000/AR1100. The remaining tabs display typical AR controller demonstrations. If using the demonstration tabs with a sensor in front of the PC screen, make sure the ARCU is maximized to ensure proper cursor alignment.

AR Configuration Utility (ARCU) User's Guide

3.1 THE CONFIGURATION TAB

FIGURE 3-2: CONFIGURATION TAB IN DIAGNOSTIC VIEW

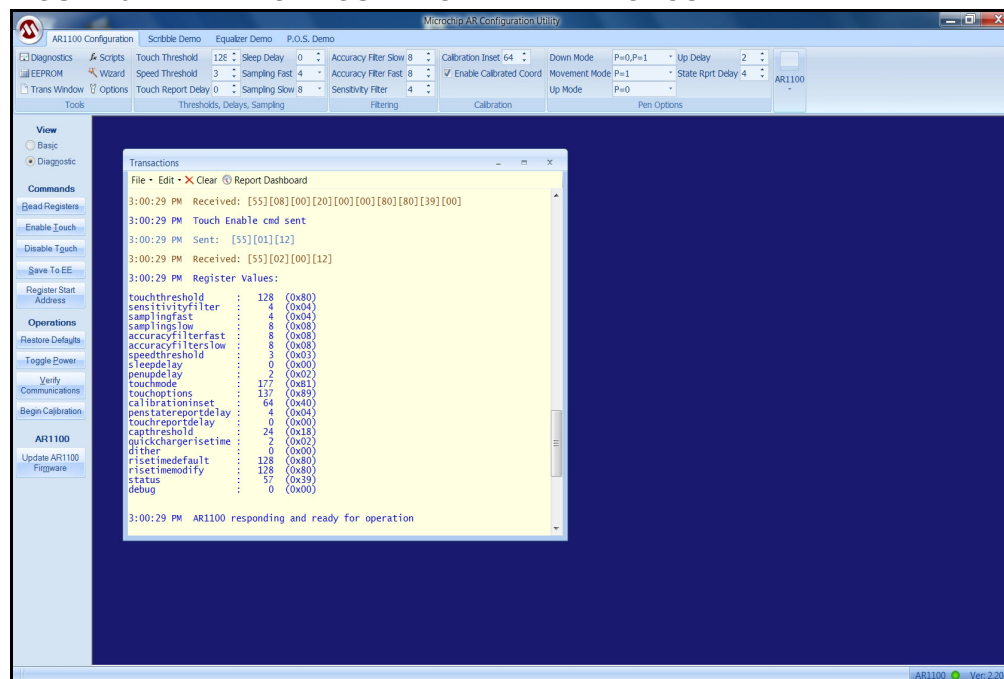

AR Configuration Utility Layout

The AR Configuration page contains all the controls necessary to configure, calibrate and test the AR1000 and the AR1100. The Configuration page consists of the following areas:

- Ribbon Tools Group
- Ribbon Registers Group
- Commands Panel
- Status Bar

Ribbon Tools Group

These are several helper windows used for various operations. Press a button to open its window. Each window's purpose is described in **Section 3.1.3 "Ribbon Tools Group"**.

Ribbon Registers Group

The AR1000/AR1100 Configuration registers are logically grouped here. You may change a register's value directly through its control. Changes are immediately written to the controller's registers and recorded in the Transactions window. Hover your mouse over a register control to see its tool tip description. See the AR1000 or AR1100 data for more detail about each register.

Commands Panel

Various firmware and utility commands and operations are located here. Results of a command/operation will be displayed in the Transaction window. Hover your mouse over a button to see its tool tip description.

Status Bar

The status bar, located at the bottom of the GUI, gives current information regarding the state of the Configuration Utility. The LED in the right corner gives the status of the AR1000 or AR1100 connection. The ARCU version number is also displayed in the right corner. The touch report rate (reports per second) is displayed in the left corner.

Note: The ribbon tools and register groups are not available in the Basic view (see Section 3.1.1 "View").

AR Configuration Utility (ARCU) User's Guide


3.1.1 View

There are two views available for the **Configuration** tab: Basic and Diagnostic. The Basic view is a subset of the Diagnostic view, hiding the parameter controls in the ribbon and displaying a simplified set of buttons on the command panel. This view is sufficient for initial exploration of the AR1000/AR1100 controllers, allowing the user to see cursor coordinates, report rates, error messages, etc. Most commands and operations are available as well (calibration, enabling/disabling the controller, reading registers, etc.).

The Diagnostic view displays the AR1000/AR1100 configuration parameters in the ribbon and allows the user to change their values, fine tuning the controller's response for a particular sensor or application.

The view can be changed via the radio buttons on the commands panel (see Figure 3-3).


FIGURE 3-3: VIEW CONTROL

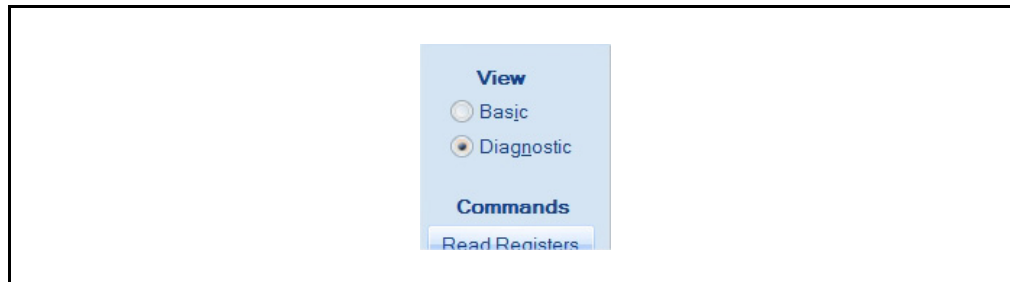

3.1.2 Commands Panel

The Commands panel holds various command and operations buttons and is located on either the right or left side of the main screen, depending on the layout preference.

FIGURE 3-4: COMMANDS PANEL

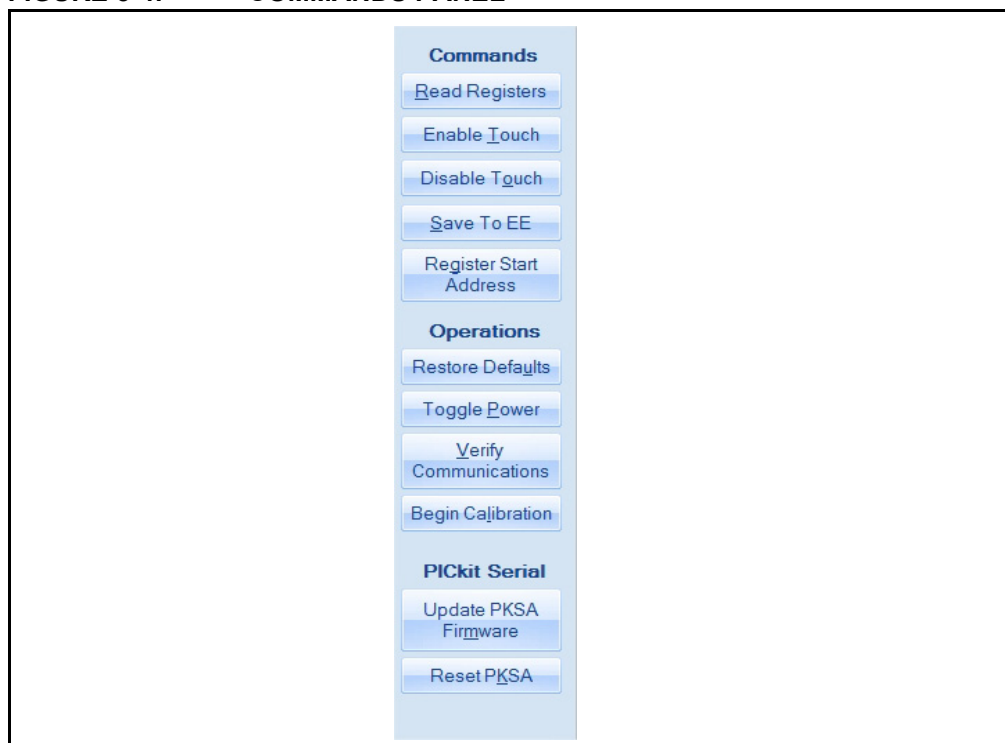

Read Registers	Reads the AR1000/AR1100 Configuration registers, displays the values on the Transactions window, and then updates the register controls in the ribbon.
Register Start Address	Sends a command to read the register start address value. See the data sheet for more information.
Enable Touch	Sends a command to enable touch for the AR1000/AR1100. If AR1000 is in I ² C or SPI mode, initiates a macro in the PICKit Serial Analyzer to read touch reports when AUX2 for I ² C or Aux1 for SPI, goes high.
Disable Touch	Sends a command to disable touch for the AR1000/AR1100. If AR1000 is in I ² C or SPI mode, disables the macro in the PICKit Serial Analyzer used to read touch reports.
Save to EE	Writes the current register values to the EEPROM and updates the displayed values in the EEPROM window (if the EEPROM window is open, see Figure 3.1.3.2).


AR Configuration Utility (ARCU) User's Guide

Restore Defaults	Restores default register values to the AR1000 by clearing the lower half of the EEPROM and power-cycling the board. Cycling power forces the AR1000 to read register and calibration values from the EEPROM. However, if EEPROM values are cleared, the AR1000 will use default register values. For the AR1100, the upper portion of the EEPROM is cleared and a Reset command is issued to force default values to be used.
Toggle Power	Reads VDD being supplied by the PICKit Serial Analyzer, turns the voltage off for 250 ms, and then restores VDD and pauses for another 250 ms. The effect of this is to power cycle the AR1000. This will force it to read register values from EEPROM. It has no effect on the AR1100.
Verify Communications	Tests communication by sending a command to read the PICKit Serial Analyzer firmware version and a command to enable touch to the AR1000. For the AR1100, this command will also read the registers.
Begin Calibration	Sends calibration command to the AR1000/AR1100. A calibration panel will appear over the entire computer screen with cross hairs appearing, showing where to touch. See the data sheet for a detailed explanation of the calibration process.
Update PKSA Firmware	(AR1000 only) Opens a file selection dialog box allowing the user to select a PICKit Serial firmware hex file. Download begins immediately after, and requires no user input. The GUI will display messages on the Transaction window right after start-up, if the firmware needs to be updated.
Update AR1100 Firmware	(AR1100 only) Opens a file selection dialog box allowing the user to select an AR1100 firmware hex file. Download begins immediately after and requires no user input. User must select first of a set of three hex files of the form xyzBoot.hex.
Reset PKSA	(AR1000 Only) Issues a Reset to the PICKit Serial Analyzer. This disables any macros running in the PICKit Serial Analyzer. This usually is only necessary if a command needs to be issued in the Scripts dialog.

3.1.3 Ribbon Tools Group

The Ribbon Tools Group contains seven windows launch buttons, of which only five or six are visible at a time, depending on the protocol being used by the ARCU.


FIGURE 3-5: TOOLS GROUP

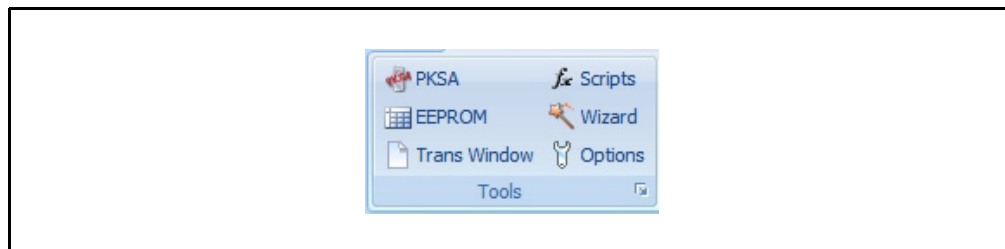

3.1.3.1 PICKit SERIAL ANALYZER WINDOW (AR1000 ONLY)

The PKSA window is used to modify the settings of the PICKit Serial Analyzer, including which communications mode (protocol) to use.

FIGURE 3-6: PICKit SERIAL ANALYZER WINDOW

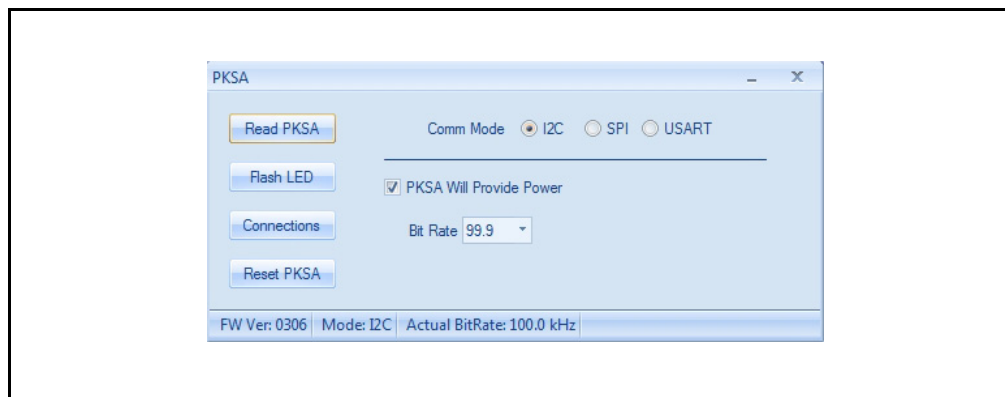

Comm Mode

Selects the method of communications your device will be using. Selecting one of the radio buttons automatically configures the PICKit Serial Analyzer, if attached, for that mode. The Communications mode is retained from run to run, and can only be changed here, in the Options menu, or in the Wizard.

I²C™

This is the default mode. When selected, you may change the bit rate by selecting a new value from the drop-down menu.

SPI

You can change the bit rate by entering in a value (in MHz) into the text box and pressing Update. The PICKit Serial Analyzer will configure its bit rate as close as possible to that value and display the actual value in the status bar below.

USART

There are no configuration options available for USART. The baud rate is fixed at 9600 to match the AR1000.

PKSA Will Provide Power


Check if you wish to have the PICKit Serial Analyzer provide a constant 5V VDD. Uncheck if you will be providing external power to the AR1000.

AR Configuration Utility (ARCU) User's Guide

Read PKSA Button	Reads the configuration of the PICkit Serial Analyzer and then proceeds to update the control and status bar.
Flash LED Button	Causes the PICkit Serial Analyzer to flash one of its LEDs (usually the 'Busy' LED) for two seconds. You can use this to verify communications with the PICkit Serial Analyzer.
Connections	Brings up a small window displaying the pin out of the PICkit Serial Analyzer for a given communication's mode.
Reset PKSA Button	Issues a Reset to the PICkit Serial Analyzer. This disables any macros running in the PICkit Serial Analyzer that would interfere with script execution. This button has the same function as the Reset PKSA button on the Command panel in the main window.
Status Bar	Displays the current configuration parameters.

3.1.3.2 EEPROM WINDOW

FIGURE 3-7: EEPROM WINDOW

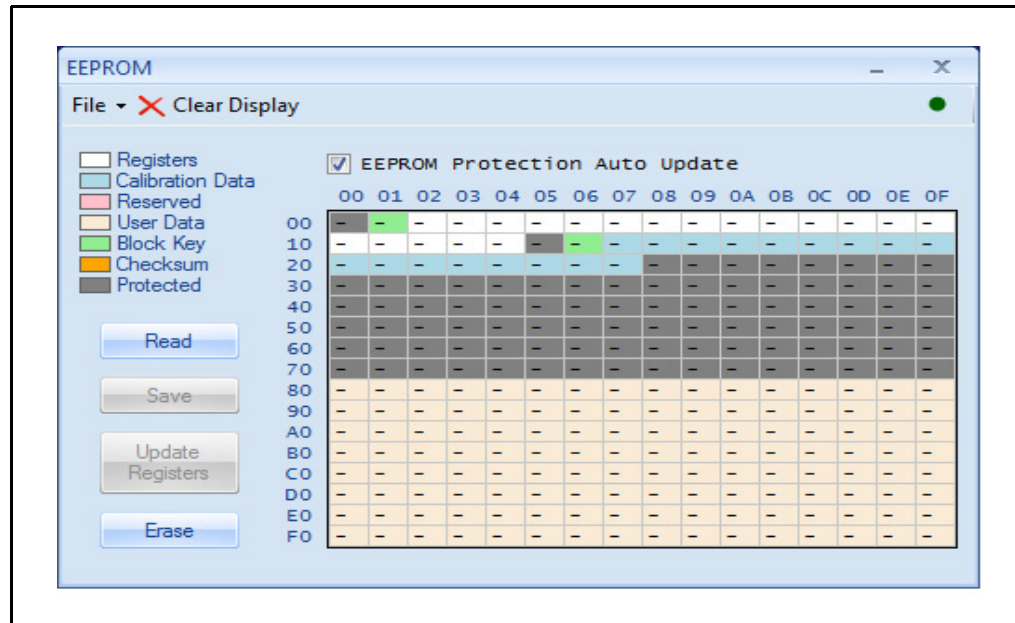

The EEPROM window is used to display and modify the contents of the AR1000 and AR1100 EEPROM. There are 256 bytes in the EEPROM, one-half of which are reserved for use by the AR1000; the other half is available for user data. Bytes 0x00-0x7F are used by the AR1000 for register and calibration data. Bytes 0x80-0xFF are free to be used by the host without risk of affecting AR1000 operation.

For the AR1011 and AR1021, an EEPROM protection checkbox has been added to prevent unexpected results when making changes to the reserved registers section.

For the AR1100, bytes 0x00-0x5F are available for user data; the rest is reserved for register and calibration data.

EEPROM 'profiles', representing the values in the window, can be saved and opened via the File menu, and then written to the EEPROM via the **Save** button.

File Menu

- Open Profile** Opens existing profiles as .txt or .csv files.
- Save Profile** Saves displayed profiles as .txt or .csv files.
- Close** Closes the window.

Clear Button Clears the contents of the window; does not erase the EEPROM.

Status LED Glows bright when the displayed values match the contents of the EEPROM; dimmed when the displayed values do not. **Note:** The EEPROM is not actively monitored by the utility; the LED is updated in response to user activity.

Read Button Reads and displays the contents of the EEPROM.

Save Button Writes the displayed values to the EEPROM.


AR Configuration Utility (ARCU) User's Guide

Update Registers Button	Writes the displayed values to the AR1000/AR1100 registers.
Erase Button	Erases the contents of the EEPROM.
Legend Keywords	
Registers	Bytes of data that set the controller's options.
Calibration Data	Bytes of data that contain information from calibration.
Reserved	These bytes are not recommended for direct modification unless directed to by support.
User Data	A section of the EEPROM that is available to end users.
Block Key	A unique byte indicating the presence of valid data in the next data block.
Checksum	An 8-bit value that is calculated by starting with the hexadecimal value 0x45, then adding all data values in the previous data block (including the block key byte) and finally using only the lower-byte portion of the calculated sum.
Protected	Non-selectable EEPROM area to protect from invalid byte data corruption.

3.1.3.3 TRANSACTIONS WINDOW

The Transactions window displays data sent to and received from the AR1000/AR1100, status and error messages, and touch report information. Sent data, received data and error messages are color-coded. The Transactions window can also display `.txt` and `.rtf` files. Multiple windows can be open at a time. However, only the most recently opened window will display real time information.

FIGURE 3-8: TRANSACTIONS WINDOW

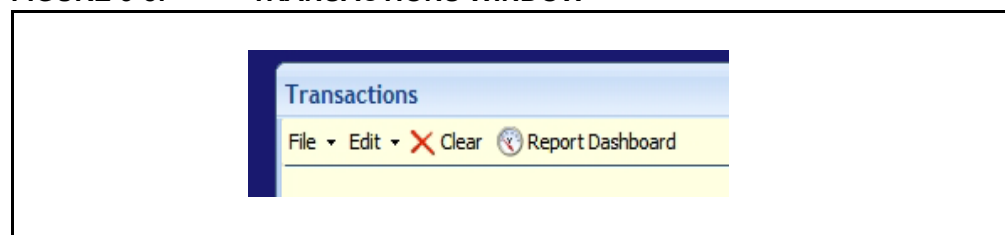

File Menu

Open	Opens existing <code>.txt</code> or <code>.csv</code> files. Automatically clears the window before opening the file.
Save	Saves contents of window as <code>.txt</code> or <code>.rtf</code> files.
Close	Closes the window.

Edit Menu

Copy	Copies selected text to the clipboard.
Paste	Pastes clipboard data to the window.
Select All	Selects all of the data in the window.

AR Configuration Utility Layout

Clear All Clears all of the contents of the window.


Clear Button Clears all of the contents of the window.

Report Dashboard Toggles the visibility of the Touch Report Dashboard. The dashboard graphically displays the touch report rate, error rate, and pen mode.

Text and images can also be typed or pasted into the Transaction window and saved along with existing content.

3.1.3.4 SCRIPTS WINDOW

FIGURE 3-9: SCRIPTS WINDOW

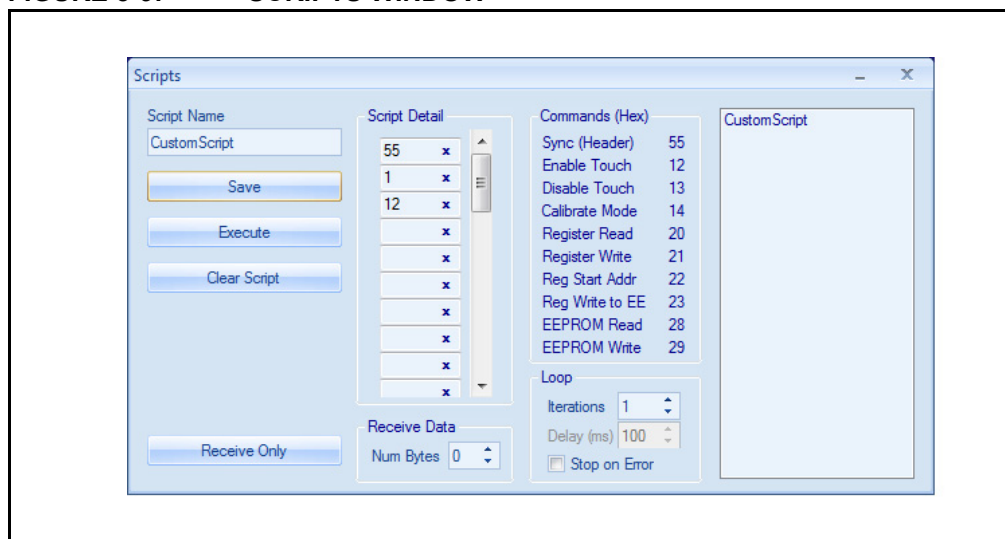

The Scripts window is used to create, save, and execute low-level commands. Script execution is protocol agnostic. That is; a given script will work with an AR1000 that is in I²C, SPI or USART mode or an AR1100 in HID-Generic mode (will not work in HID-Digitizer or HID-Mouse mode). To load an existing script, double click the script name in the far right panel.

A script consists of two parts; data sent to the AR1000/AR1100 (bytes contained in the Script Detail box), and data received in response (expected byte count displayed in the Receive Data box). Data sent by and received in response to the script are displayed in the Transactions window.

To create a script, enter a name you wish to give it, fill in the bytes in the Script Detail, and then enter the expected return byte count in the Receive Data. Save or execute the script by pressing the appropriate button. For more information about the format of the commands, see the AR1000 or AR1100 data sheet.

Script Name When creating a script, enter a name here.

Save Button Saves the script.

Execute Button Executes the displayed script. Results are shown in the Transactions window. **Note:** For the AR1000, it may be necessary to press the **Reset PKSA** button to disable any running macros before a script can be run.

Clear Script Button Clears the current script.

AR Configuration Utility (ARCU) User's Guide

Reset PKSA Button	(AR1000 only) Issues a Reset to the PICkit Serial Analyzer. This disables any macros running in the PICkit Serial Analyzer that would interfere with script execution. This button has the same function as the Reset PKSA button on the Command panel in the main window.
Receive Only Button	Issues a read command without sending a script. This aids in low-level testing of the AR1000/AR1100 by allowing you to separate the sending of a script (with 0 expected bytes) and at a later time, issue a Read command.
Script Detail	Enter the body of the script here. By default, bytes are interpreted as hexadecimal. If you wish to use decimal, click on the 'x' in a byte text box to change the radix. Data displayed in the Transactions window is always hexadecimal, however.
Receive Data	Enter the number of expected return bytes.
Commands	Command values; see the AR1000 and AR1100 data sheets for information about a specific command.
Loop	Scripts may be run for a specified number of iterations, or to run continuously, enter an iteration value of 0. The delay between each iteration may be adjusted by the Delay control. Check the Stop on Error checkbox to stop script iteration in the event of an error.
Script Panel	The far right panel contains the names of existing scripts. Double click the script name to load it.

3.1.3.5 WIZARD

FIGURE 3-10: CONFIGURATION WIZARD



The Wizard steps you through the process of configuring and calibrating the AR1000 and the AR1100. If you are using the touch sensor provided in one of the development kits, print out the calibration template before beginning the Wizard. You will use this template to provide a standard set of calibration points so the movement of your finger or stylus on the touch sensor is correctly reflected on your computer monitor.

Depending on your choices in the Wizard, you may be asked to install an optimized Configuration register profile. This profile will be written to the EEPROM and reflected in the Configuration registers after completion of the Wizard.


The Wizard is one of the choices you have the first time you run the Configuration Utility. However, you may run the Wizard again at any time by pressing the **Wizard** button, located in the Tools group in the ribbon in the Diagnostic view, or on the Commands Panel when in Basic view.

AR Configuration Utility (ARCU) User's Guide

3.1.3.6 OPTIONS WINDOW

The Options window contains several parameters that allow you to configure the GUI for your particular work environment. Some options are strictly for cosmetic purposes, while others allow you to fine tune the GUI according to your computer's capabilities. All options are retained from run to run.

FIGURE 3-11: OPTIONS WINDOW

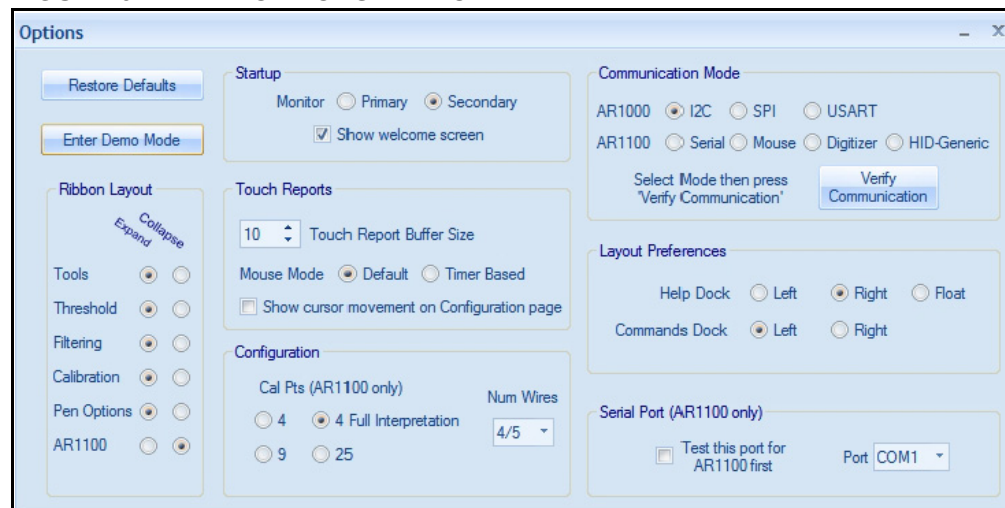

Restore Defaults

Changes all options to their default values.

Enter Demo Mode

Hides the **Configuration** tab and launches the Scribble demo. Use this mode to ensure that users do not have access to the Configuration registers.

Ribbon Layout

Allows you to expand or collapse groupboxes within the ribbon. Depending on the size of your monitor, you may want to collapse groupboxes you do not use often. If a groupbox is collapsed, you can still access its controls by clicking on it. Its controls will appear below the ribbon.

Startup Monitor

Allows you to choose in which monitor the application will start. This option has no effect if there is no secondary monitor present.

Show Welcome Screen

Check this if you want the welcome screen to open the next time you start the GUI.

AR Configuration Utility Layout


Touch Report Buffer Size	A typical touch sensor transmits 100 to 200 touch reports per second. Computers with slower processors may have a difficult time keeping up with the higher rates, so touch reports are buffered by the GUI and sent in mass to the Transaction window to reduce CPU usage. Increasing this value sends updates to the Transaction window less frequently and decreases CPU usage. Decreasing this value updates the Transaction window more often, but increases CPU usage. While the effect of faster updates is a more fluid response in the Transaction window, the potential danger is dropped bytes in the touch reports due to CPU starvation. A good rule of thumb is to keep your CPU performance under 50% while touch reports are being generated, as indicated by the Windows Task Manager.
Mouse Mode	This option allows you to change how the GUI interprets pen up, down, and move touch reports for the AR1000. The default method examines the contents of each report to make this determination based on the default values used in the TouchMode register. Important: If the default values are not used in the TouchMode register, this method will not work. Use the Timer-based method if default values are not used in the TouchMode register. This mode simply sends mouse move events, if touch reports are being generated and sends a mouse up event 100 milliseconds after the last touch report.
Show Cursor – Config	Allows the cursor to be controlled by touch reports on the Configuration page. Checking this option will make it easier to navigate between the Configuration and other tabs, but may make it confusing while using touch reporting on the Configuration page. For example, while pressing on the Transactions window, the window may behave strangely because it is trying to update touch reports in real time, but also trying to insert the cursor wherever the touch is being generated. It is recommended this option be left unchecked until you become familiar with the operation of the AR Configuration Utility.
4, 9, 25 Point Calibration	Choose how many points to use during the calibration procedure. More points will provide more accurate tracking, especially for larger sensors. This option only applies to the AR1100.
Num Wires	Select the number of wires your sensor has. The option for 5-wire is not shown since it is hardware selectable on both the AR1000 and AR1100 and will override 4 or 8-wire selection.
Communication Mode	Select which communication protocol you wish to use. Press the Test Communication button to initialize the protocol in the GUI and attempt communication with the AR1000/AR1100.

AR Configuration Utility (ARCU) User's Guide

Help Dock	Allows you to dock the Help panel to the left or right of the GUI, or float it as a separate window.
Commands Dock	Allows you to dock the Commands panel to the left or right of the GUI.
Test This Port First	Tells the GUI to try using the port in the accompanying drop down first when attempting to find the AR1100. The default method is to loop through all available COM ports until the AR1100 is found, but this has the possibility of changing port settings for another device that is attached.
COM Port	Lists all available COM ports found on the PC. Select the one to which the AR1100 is attached and check the box to the left to have the GUI test communications on that port before others.

3.1.3.7 DIAGNOSTICS WINDOW (AR1100 ONLY)

FIGURE 3-12: DIAGNOSTICS WINDOW

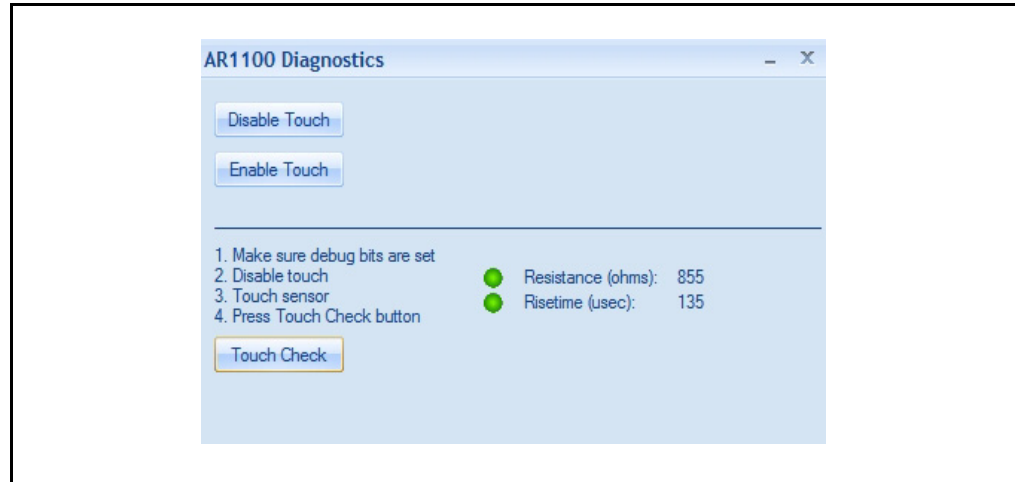

The Diagnostics window is used to run resistance and risetime diagnostics. In order for a diagnostic to be generated, the following bits must be set in the AR1100 groupbox in the ribbon:

Resistance: Risetime Resistance (RTR)

Risetime: Risetime (RT)

To generate a diagnostic, perform the following steps:

- Set the above bits.
- Disable touch.
- Touch and hold the sensor.
- Press the **Touch Check** button.
- Enable touch when finished.

The measured resistance and risetime will be displayed in the Diagnostics window and the Transactions window will display the commands and responses.

Disable Touch Button Disables touch response on the AR1100. Touch must be disabled prior to running a diagnostic.

Enable Touch Button Enables touch response on the AR1100. You should enable touch after running a diagnostic.

Touch Check Button Sends a `TOUCH_CHECK` command to the AR1100.

Diagnostic Results A LED will glow bright green if the diagnostic was successfully measured and the value will be displayed along side.

AR Configuration Utility (ARCU) User's Guide

3.1.4 Keyboard Shortcuts

The AR Configuration Utility has a multitude of keyboard shortcuts that can be used to quickly navigate through the utility. The included keyboard shortcuts are:

Help Press the F1 key to bring up the Help menu

Command Bar Shortcuts To use the command bar keyboard shortcuts, first press the ALT key, followed immediately by the underlined character on the command bar. The letter is capitalized here for visibility.

View:

Basic View	ALT + I (I as in Icarus)
Diagnostic View	ALT + N

Commands:

Read Registers	ALT + R
Enable Touch	ALT + T
Disable Touch	ALT + O
Save To EE	ALT + S
Register Start Address	ALT + G

Operations:

Restore Defaults	ALT + U
Toggle Power	ALT + P
Verify Communications	ALT + V
Begin Calibration	ALT + L
Start Wizard	ALT + W

AR1000/AR1100:

Update PKSA Firmware	ALT + M
Reset PKSA	ALT + K

Ribbon Shortcuts To use the ribbon keyboard shortcuts, first press and release the ALT button on the keyboard followed by pressing two keys according to the list below.

Tools Ribbon:

Diagnostics Window	ALT, A, D
EEPROM Window	ALT, A, E
Trans Window	ALT, A, T
Scripts Window	ALT, A, S
Wizard	ALT, A, W
Options	ALT, A, O

AR Configuration Utility Layout

Thresholds, Delays, Samplings:

Touch Threshold	ALT, B, T
Speed Threshold	ALT, B, S
Touch Report Delay	ALT, B, D
Sleep Delay	ALT, B, P
Sampling Fast	ALT, B, F
Sampling Slow	ALT, B, W

Filtering:

Accuracy Filter Slow	ALT, C, W
Accuracy Filter Fast	ALT, C, F
Sensitivity Filter	ALT, C, S

Calibration:

Calibration Inset	ALT, D, I (I as in Icarus)
Enable Calibrated Coord	ALT, D, E

Pen Options:

Down Mode	ALT, E, D
Movement Mode	ALT, E, M
Up Mode	ALT, E, U
Up Delay	ALT, E, P
State Rprt Delay	ALT, E, R

AR1100:

EEV	ALT, F, E
CALV	ALT, F, L
DRT	ALT, F, D
RiseTm Default	ALT, F, T
RiseTm Modify	ALT, F, M
Dither	ALT, F, I
TEN	ALT, F, N
Verbose Cal	ALT, F, V
DRT	ALT, F, W
RT	ALT, F, R
RTC	ALT, F, C
RTR	ALT, F, A
SP	ALT, F, S
MM	ALT, F, B

AR Configuration Utility (ARCU) User's Guide

3.2 THE DEMONSTRATION TABS

The AR Configuration Utility has three tabs that can be used to test and demonstrate touch movements: **Scribble**, **Equalizer**, and **Point-Of-Sale** (P.O.S.).


Ideally, you would have a touch sensor the same size as your computer monitor, or laptop screen. Then you could place it over your monitor, attach it to the AR1000/AR1100, perform a calibration and use your finger or stylus to interact with the controls on the demonstration tabs.

The AR1000 and AR1100 development kits provide an alternative method. You can use the included touch sensor along with the templates provided in the Calibration Template document to interact with the P.O.S. demonstration tab controls. Print out the Calibration Template document (be sure to print it at 100% of its size) and run the Wizard located in the Tools group of the ribbon. If you have already run the Wizard and have established communications with the AR1000/AR1100, you can also simply press the **Begin Calibration** button on the Commands panel and use the template instead of the on-screen calibration points.

Note: When interacting with the demonstration tabs via the templates, the AR1000/AR1100 will also control your cursor; however, your mouse is still usable.

3.2.1 Scribble Tab


FIGURE 3-13: SCRIBBLE TAB

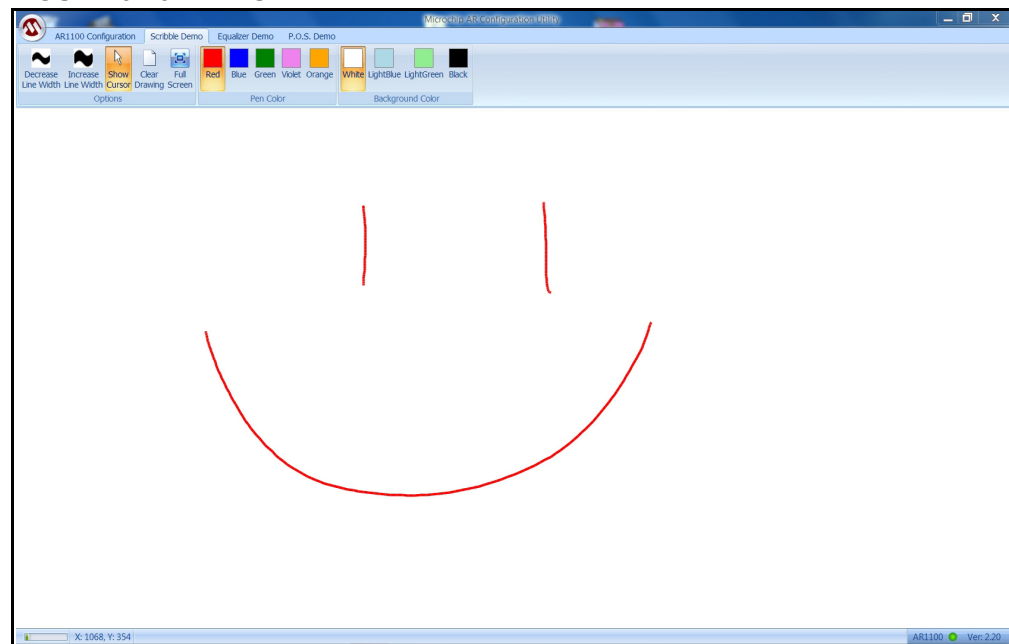

Scribble is a simple drawing program. The first set of buttons allows you to increase or decrease the width of the pen, clear the drawing, and enter Full Screen mode. The second set of buttons allows you to change the color of the pen. The third set of buttons allows you change the background color. You also have the option of using this tab in Full Screen mode.

3.2.2 Equalizer Tab

FIGURE 3-14: EQUALIZER TAB

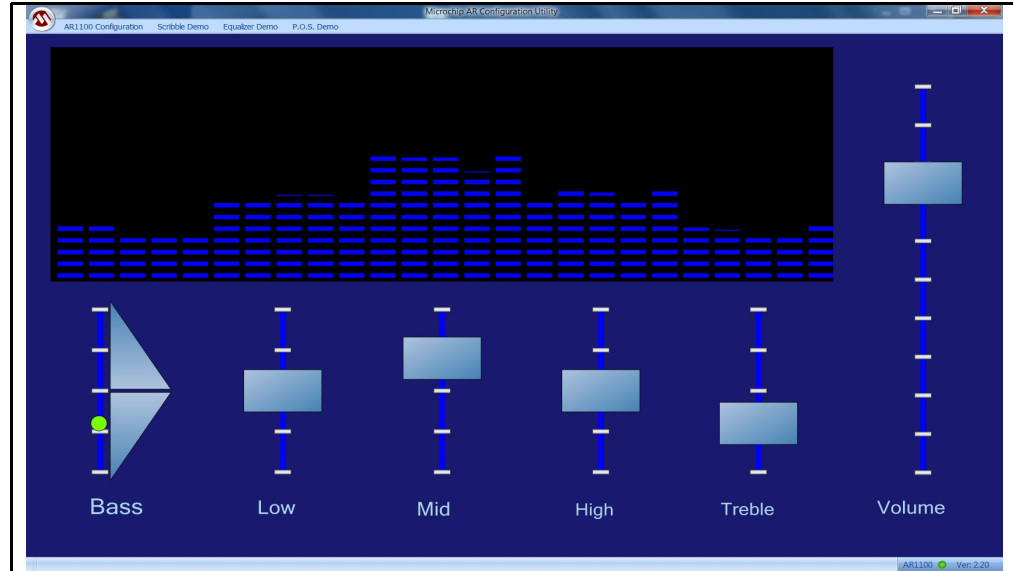


This tab demonstrates a graphic equalizer. All of the controls are sliders, except the Bass control. To change the Bass control setting, press and hold the **Up** or **Down** button next to the scale.

The Bass control is different than the rest to demonstrate an alternative technique that achieves the same result. Press the **Up/Down** buttons to change the equalizer value.

3.2.3 P.O.S. Tab

FIGURE 3-15: P.O.S TAB

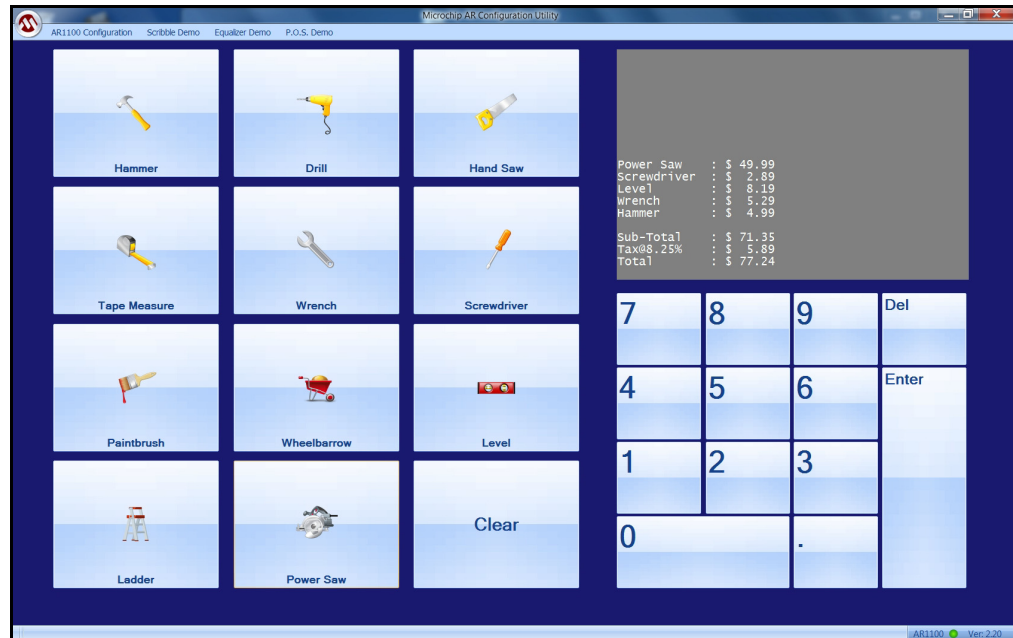

This is a point-of-sale emulator. Press an item button to add that item to your shopping list. The **Clear** button resets your shopping list. There is also a numeric keypad for entering unspecified items.

AR Configuration Utility (ARCU) User's Guide

NOTES:

AR Configuration Utility (ARCU) User's Guide

NOTES:

Worldwide Sales and Service

AMERICAS

Corporate Office
2355 West Chandler Blvd.
Chandler, AZ 85224-6199
Tel: 480-792-7200
Fax: 480-792-7277
Technical Support:
<http://www.microchip.com/support>
Web Address:
www.microchip.com

Atlanta
Duluth, GA
Tel: 678-957-9614
Fax: 678-957-1455

Boston
Westborough, MA
Tel: 774-760-0087
Fax: 774-760-0088

Chicago
Itasca, IL
Tel: 630-285-0071
Fax: 630-285-0075

Cleveland
Independence, OH
Tel: 216-447-0464
Fax: 216-447-0643

Dallas
Addison, TX
Tel: 972-818-7423
Fax: 972-818-2924

Detroit
Farmington Hills, MI
Tel: 248-538-2250
Fax: 248-538-2260

Indianapolis
Noblesville, IN
Tel: 317-773-8323
Fax: 317-773-5453

Los Angeles
Mission Viejo, CA
Tel: 949-462-9523
Fax: 949-462-9608

Santa Clara
Santa Clara, CA
Tel: 408-961-6444
Fax: 408-961-6445

Toronto
Mississauga, Ontario,
Canada
Tel: 905-673-0699
Fax: 905-673-6509

ASIA/PACIFIC

Asia Pacific Office
Suites 3707-14, 37th Floor
Tower 6, The Gateway
Harbour City, Kowloon
Hong Kong
Tel: 852-2401-1200
Fax: 852-2401-3431

Australia - Sydney
Tel: 61-2-9868-6733
Fax: 61-2-9868-6755

China - Beijing
Tel: 86-10-8569-7000
Fax: 86-10-8528-2104

China - Chengdu
Tel: 86-28-8665-5511
Fax: 86-28-8665-7889

China - Chongqing
Tel: 86-23-8980-9588
Fax: 86-23-8980-9500

China - Hangzhou
Tel: 86-571-2819-3187
Fax: 86-571-2819-3189

China - Hong Kong SAR
Tel: 852-2401-1200
Fax: 852-2401-3431

China - Nanjing
Tel: 86-25-8473-2460
Fax: 86-25-8473-2470

China - Qingdao
Tel: 86-532-8502-7355
Fax: 86-532-8502-7205

China - Shanghai
Tel: 86-21-5407-5533
Fax: 86-21-5407-5066

China - Shenyang
Tel: 86-24-2334-2829
Fax: 86-24-2334-2393

China - Shenzhen
Tel: 86-755-8203-2660
Fax: 86-755-8203-1760

China - Wuhan
Tel: 86-27-5980-5300
Fax: 86-27-5980-5118

China - Xian
Tel: 86-29-8833-7252
Fax: 86-29-8833-7256

China - Xiamen
Tel: 86-592-2388138
Fax: 86-592-2388130

China - Zhuhai
Tel: 86-756-3210040
Fax: 86-756-3210049

ASIA/PACIFIC

India - Bangalore
Tel: 91-80-3090-4444
Fax: 91-80-3090-4123

India - New Delhi
Tel: 91-11-4160-8631
Fax: 91-11-4160-8632

India - Pune
Tel: 91-20-2566-1512
Fax: 91-20-2566-1513

Japan - Osaka
Tel: 81-66-152-7160
Fax: 81-66-152-9310

Japan - Yokohama
Tel: 81-45-471-6166
Fax: 81-45-471-6122

Korea - Daegu
Tel: 82-53-744-4301
Fax: 82-53-744-4302

Korea - Seoul
Tel: 82-2-554-7200
Fax: 82-2-558-5932 or
82-2-558-5934

Malaysia - Kuala Lumpur
Tel: 60-3-6201-9857
Fax: 60-3-6201-9859

Malaysia - Penang
Tel: 60-4-227-8870
Fax: 60-4-227-4068

Philippines - Manila
Tel: 63-2-634-9065
Fax: 63-2-634-9069

Singapore
Tel: 65-6334-8870
Fax: 65-6334-8850

Taiwan - Hsin Chu
Tel: 886-3-5778-366
Fax: 886-3-5770-955

Taiwan - Kaohsiung
Tel: 886-7-536-4818
Fax: 886-7-330-9305

Taiwan - Taipei
Tel: 886-2-2500-6610
Fax: 886-2-2508-0102

Thailand - Bangkok
Tel: 66-2-694-1351
Fax: 66-2-694-1350

EUROPE

Austria - Wels
Tel: 43-7242-2244-39
Fax: 43-7242-2244-393

Denmark - Copenhagen
Tel: 45-4450-2828
Fax: 45-4485-2829

France - Paris
Tel: 33-1-69-53-63-20
Fax: 33-1-69-30-90-79

Germany - Munich
Tel: 49-89-627-144-0
Fax: 49-89-627-144-44

Italy - Milan
Tel: 39-0331-742611
Fax: 39-0331-466781

Netherlands - Drunen
Tel: 31-416-690399
Fax: 31-416-690340

Spain - Madrid
Tel: 34-91-708-08-90
Fax: 34-91-708-08-91

UK - Wokingham
Tel: 44-118-921-5869
Fax: 44-118-921-5820